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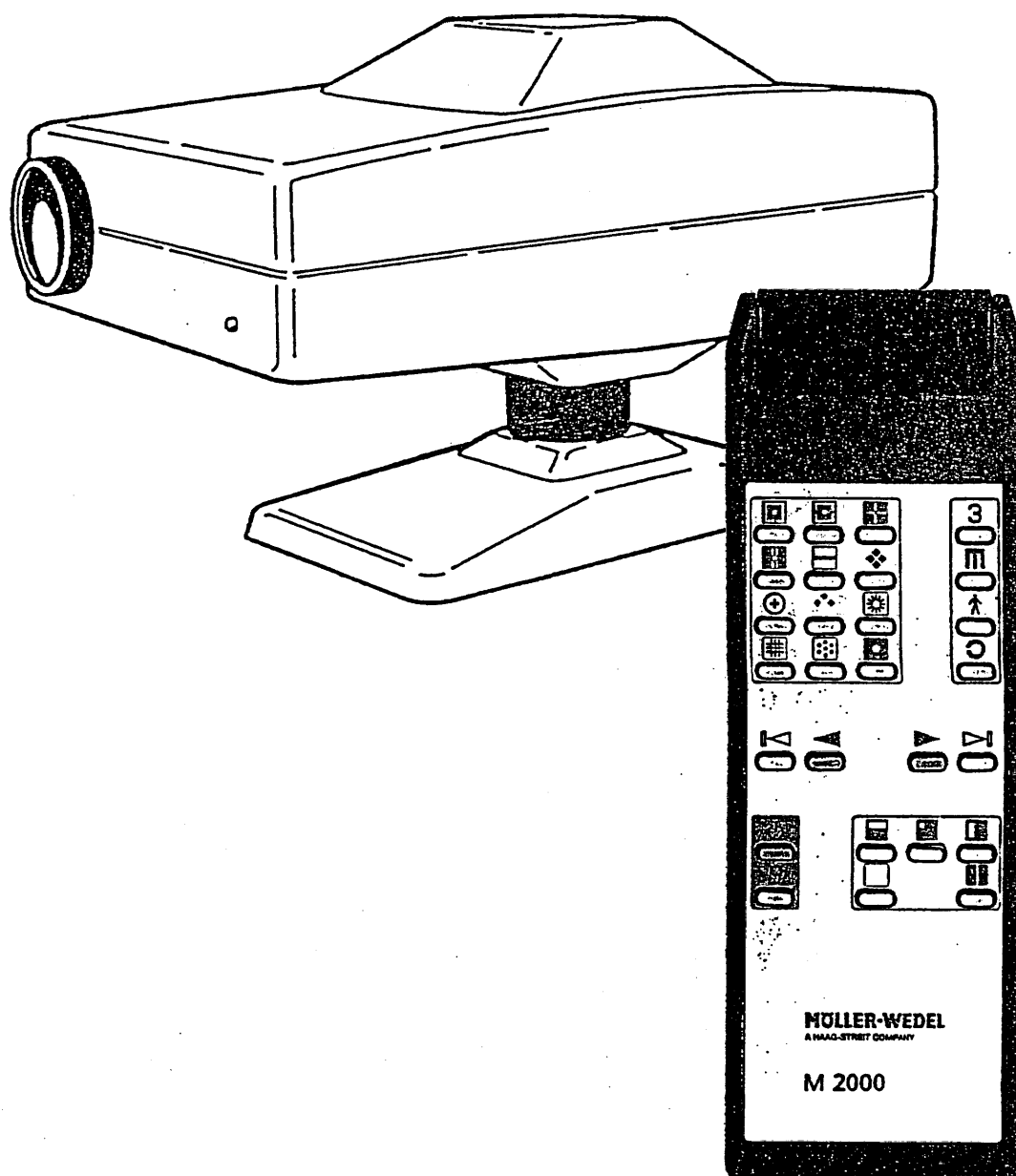
Operating Manual

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Test Type Projector  
M 2000

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## 1. WARRANTY AND LIMITATION OF LIABILITY

MÖLLER-WEDEL warrants that the equipment is free from manufacturer defects in material or workmanship for a period of one year from the date of original purchase when used in accordance with MÖLLER-WEDEL's manual.

In case of any defect during the warranty period, customer shall give MÖLLER-WEDEL prompt notice and MÖLLER-WEDEL will repair or replace the equipment at its option. This warranty shall be void if the equipment is modified or serviced by persons not authorized by MÖLLER-WEDEL, in case of improper maintenance (in the case of maintenance not carried out by MÖLLER-WEDEL), or in the case of improper handling of the equipment.

**THE FOREGOING IS A LIMITED WARRANTY AND MÖLLER-WEDEL MAKES AND CUSTOMER RECEIVES NO OTHER WARRANTY EXPRESSED OR IMPLIED AND ALL WARRANTIES OR IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE ARE EXPRESSLY EXCLUDED.**

This express warranty is in lieu of all other liabilities and obligations of MÖLLER-WEDEL with respect to the equipment. In no event shall MÖLLER-WEDEL be liable to customer for any claims for damages whether direct, indirect, incidental, special, or consequential, including but not limited to lost business and lost profits whether foreseeable or not even if MÖLLER-WEDEL has been advised of the possibility of such damages.

### Special important information for Technical Services

All installations, repairs, modifications, maintenance, or other technical services must be carried out by personnel who are trained and explicitly authorized by MÖLLER-WEDEL for such work. Only original parts from MÖLLER-WEDEL must be used for maintenance or repairs.

Installations, maintenance, and repairs must be carried out according to technical manuals provided by MÖLLER-WEDEL. Modifications must be approved by the factory. After all technical services the equipment must be tested and re-adjusted according to MÖLLER-WEDEL's technical instructions.

For technical inquiries the part number should be indicated. When parts are replaced, added, or removed the laws or regulations concerning medical equipment (e.g. traceability, approvals for added parts, electrical safety, etc.) have to be considered. In any case of doubt please contact MÖLLER-WEDEL for clarification.

### Notes

Year of manufacturing and serial number of the unit: Refer to the identification label.

Keep the Operating Manual for later use.

### Safety Notes

The unit is only permitted to be operated according to this Operating Manual also including the destined use. For user's and patient's safety the maintenance instructions according to this Operating Manual should be followed, the minutes of which have to be written down and stored. Although no regular maintenances are required we recommend to perform the yearly preventive inspections suggested by MÖLLER-WEDEL.

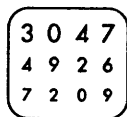
Under all circumstances MÖLLER-WEDEL or a person or a company authorized by MÖLLER-WEDEL should be addressed at least once within 3 years for information concerning safety measures.

<b>Inhalt</b>	<b>Seite</b>	<b>Contents</b>	<b>Page</b>
<b>1. Wichtige Hinweise .....</b>	<b>2</b>	<b>1. Warranty and Limitation of Liability .....</b>	<b>2</b>
<b>2. Bestimmungsgemäße Verwendung .....</b>	<b>5</b>	<b>2. Intended Use .....</b>	<b>5</b>
<b>3. Funktionsbeschreibung .....</b>	<b>5</b>	<b>3. Description of Functions.....</b>	<b>7</b>
<b>4. Montage.....</b>	<b>7</b>	<b>4. Installation .....</b>	<b>7</b>
<b>4.1 Fernbedienung.....</b>	<b>7</b>	<b>4.1 Remote Control.....</b>	<b>7</b>
<b>5. Inbetriebnahme .....</b>	<b>9</b>	<b>5. Start Up .....</b>	<b>9</b>
<b>6. Betrieb beenden.....</b>	<b>9</b>	<b>6. End of Operation .....</b>	<b>9</b>
<b>7. Bedienung .....</b>	<b>9</b>	<b>7. Operation .....</b>	<b>9</b>
<b>8. Lampenwechsel .....</b>	<b>15</b>	<b>8. Exchange of Lamps.....</b>	<b>15</b>
<b>9. Pflege.....</b>	<b>15</b>	<b>9. Care .....</b>	<b>15</b>
<b>10. Störungsbeseitigung .....</b>	<b>17</b>	<b>10. Trouble Shooting and Remedy.....</b>	<b>17</b>
<b>11. Technische Daten .....</b>	<b>17</b>	<b>11. Technical Data .....</b>	<b>17</b>
 <b>Anhang:</b>		 <b>Annex:</b>	
<b>CE EG - Konformitätserklärung</b>		<b>CE EC - Declaration of Conformity</b>	

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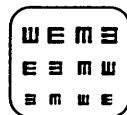
Landolt rings referring to size, quantity, spaces from each other and to the edges according to DIN 58220 and ISO 8596 in v. a. values

0,1	0,125	0,16	0,2	0,25	0,32	0,4	0,5	0,63
0,7	0,8	1,0	1,25	1,6	2,0			

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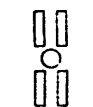
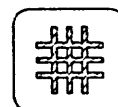
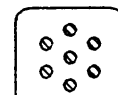
## 2. Intended Use

The Test Type Projector M 2000 is designed for projection of test types to determine the visual acuity. The projector should only be operated by specialists after proper instruction according to the operating manual.

The test type projector is part of a refraction system.

The conditions of installation and operation of the equipment have to correspond to the following requirements:

- Minor vibration
- Careful handling
- Clean environment
- Avoidance of extreme mechanical stress

## 3. Description of Functions

The M 2000 projects test types on to an aluminum projection screen.

The tests are switched IR-remote controlled. On a full frame field special images or test types are positioned in one, two, or three rows of different acuity values. Test types (numbers, E-hooks, children pictures) are blocked by masks.

The projected image is shown on a projector display marked by a symbol and the acuity value.

The projector may be connected to the phoropter VISUTRON PLUS via interface. Depending on the selected test, the other equipment will automatically be matched

Availability of test types: Refer to page 4

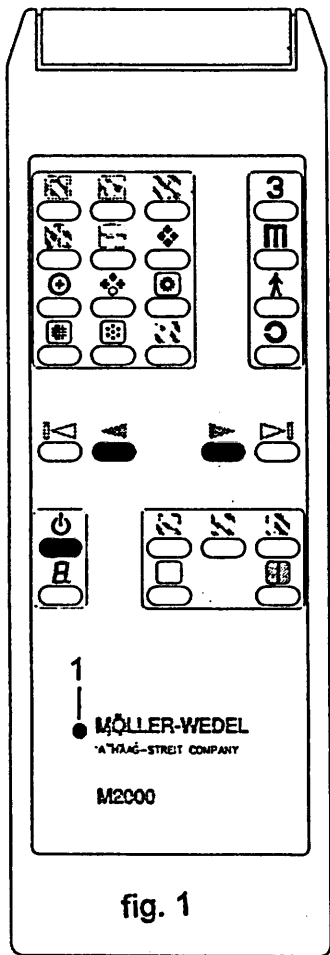
fig. 1

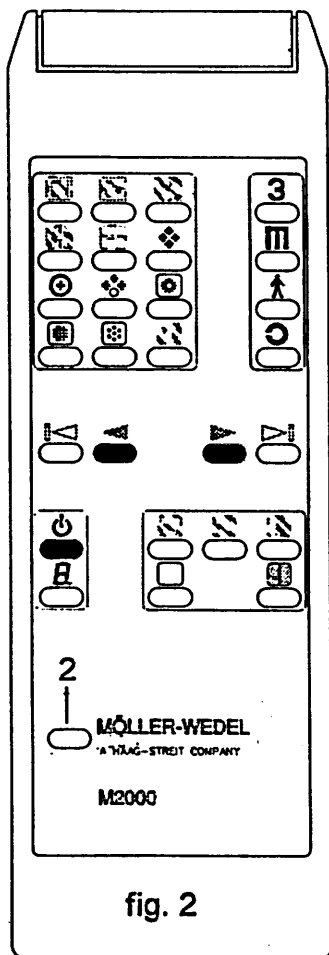
fig. 2

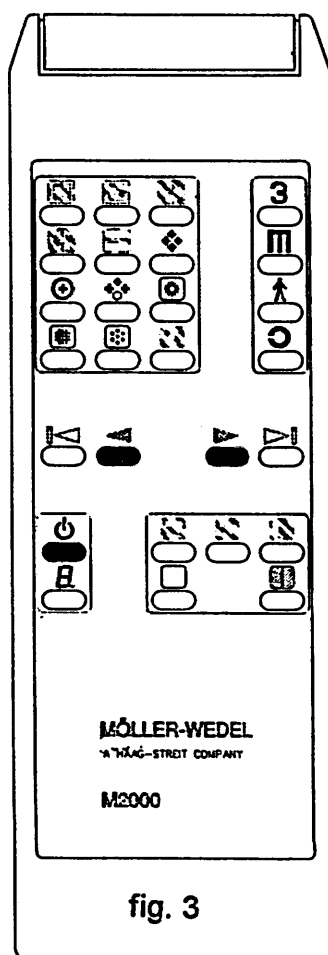
fig. 3

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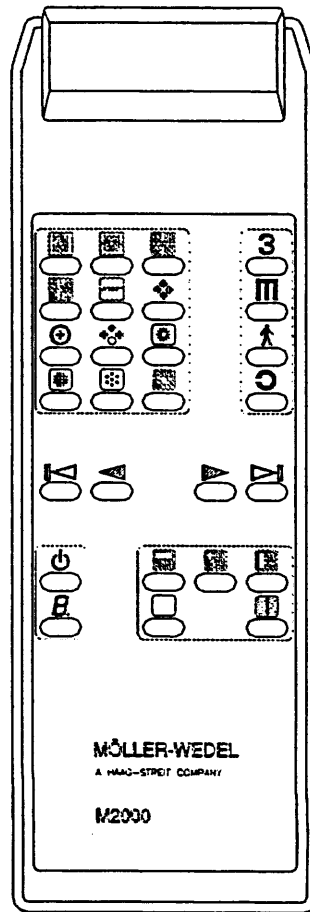
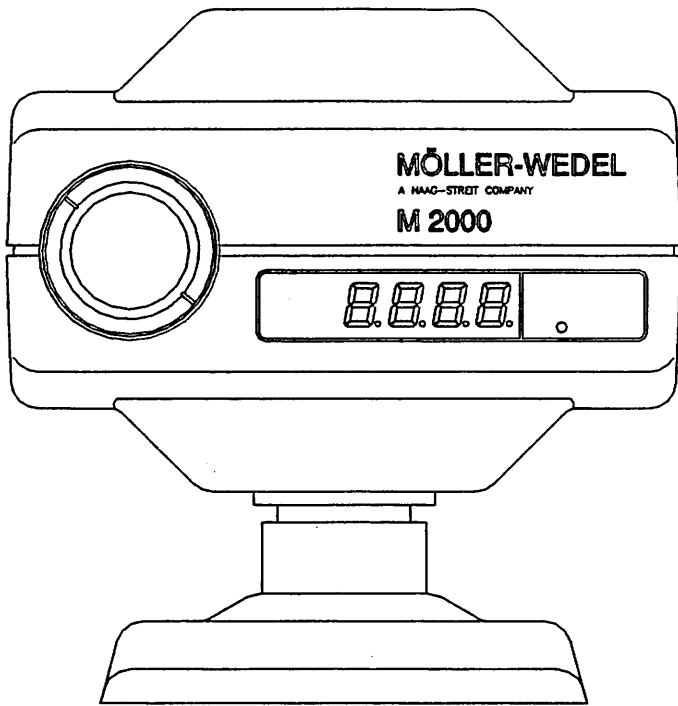
## 4. Installation

- Open the cardboard.
- Take off the projection screen.
- Take off the projector.
- Take off cable, remote control, and adapter from the cardboard located laterally.
- Depending on the diameter of the mount for the projector (wall arm or refraction unit) an adapter may be required.
  
- Attachment of the adapter:  
Loosen two Allen screws outside the projector mount by Allen key, width 2.5 mm.  
Remove the inner part.  
Insert the adapter and tighten the same screws.
  
- Install the projector.  
Connect the mains cable. Make sure that the voltage on the projector corresponds to the local voltage!
- For coupling to the VISUTRON PLUS connect the data line to the interface in the central unit of the VISUTRON PLUS.

### 4.1 Remote Control

- Insert the batteries into the hand switch: Shift the cover into direction of the arrow and lift it.  
Position the batteries following the engraved instruction. Close the cover.
- Programming of the remote control :  
**Fig. 1** Depress the button via pin.  
  
**Fig. 2** Depress the button.  
**Fig. 3** Remove all batteries.  
Shortcircuit the upper contacts (at the side of the sender) e.g. by forcettes or clip. Insert the batteries following the engraved instruction.  
Locate the cover and push it to the opposite direction of the arrow until it locks in.
- Multiple programming:  
If several M 2000 projectors shall be operated in one room, please call for the service.

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## 5. Start Up

- Switch on the instrument.
- Adjust the projector to the projection screen by rotating, tilting, or inclining.
- By pulling or pushing the objective lens focus the images on to the projection screen.

The lamp will automatically be switched off after a few seconds. The moving lights in the display indicate the 'Sleep' mode. When switching on again the last test field during switching off will be projected again.

## 6. End of Operation

- Switch off the instrument. In general this will be done via the refraction unit.

## 7. Operation

Buttons and marks:

Display	Buttons	
SP. 1		Aniseikonia test vertical
SP. 2		Aniseikonia test horizontal
SP. 3		Heterophoria test
SP. 4		Stereo test
SP. 5		Balance test
SP. 6		Bichrome balance test
SP. 7		Schober test
SP. 8		Astigmatism test
SP. 9		Worth test
SP. 10		Grid target test
SP. 11		Dots
SP. 12		Fixation dot
		Numbers
		E hooks
		Children's tests
		Landolt rings
		Backwards (larger images)
		(3 steps backwards)
		Forwards (smaller images)
		(3 steps forwards)
		ON/OFF halogen lamp
		ON/OFF display on projector
		Horizontal row
		Single images
		Vertical row
		Full frame field
		Red/green filter

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A few seconds after switching on, the lamp will be lit. The first projected test is the row with numbers for visual acuity 0.4 or the visual acuity step programmed according to personal requirements. A beep signal indicates the end of a test row or the selected special image being reached. When interchanging between menus, numbers, E-hooks, children's pictures, and Landolt rings the visual acuity will remain the same. Special images will directly be selected. When changing from special images to the menu, the pre-programmed visual acuity will be set.

### Programming of the acuity steps of M 2000 via remote control

Programming is only applicable if the setting which was made factory-side shall be changed. If there is no data exchange between M 2000 and VISUTRON PLUS, the items 1-3 and 7-9 are not applicable.

The following steps have to be performed successively and without a pause:

1. Switch off the projector at its rear next to the mains plug.
2. Pull out the data line to VISUTRON PLUS.
3. Switch on the projector at its rear next to the mains plug and wait until a test type appears on the screen.
4. Switch off the projector via button  on the remote control.
5. Successively depress the following buttons:  
  
The projector switches to SET UP mode.

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6. At the left side of the projector display a symbol appears which indicates the corresponding kind of test types.

At the right side next to the symbol, the acuity values can be set to the desired acuity value via arrow buttons.

By depressing the button  a smaller acuity value (= large test types) will be set.

By depressing the button  a larger acuity value (= small test types) will be set.

**After setting the acuity value or if no alteration of the acuity value is required, the setting has to be confirmed via button  - .**

The next symbol appears for which the acuity value can now be set. This procedure has to be repeated for the different kinds of test types. Then a „t“ appears with a minute indication which shows after what period the lamp will be switched off (sleep-mode). This period can be changed via arrow buttons. Finally '3-3' appears in the display of the projector. Confirm this function via  - button.

The projector returns to start position.

Succession of appearing symbols for test types on the projector:

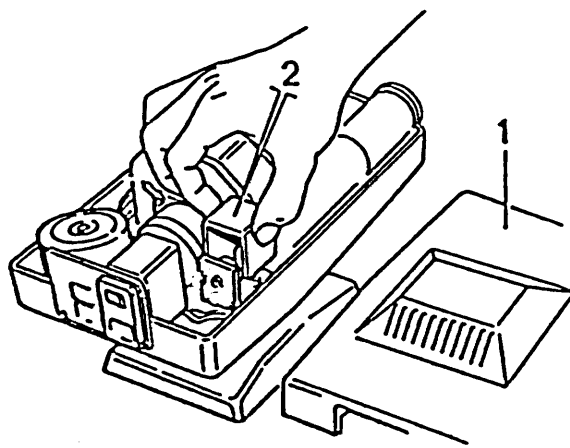
- 3 means numbers (for acuity value after switching on)

C	means Landolt rings	} for acuity values after changing from special test types to other test types
3	means numbers	
E	means E-hooks	
S	means children's pict.	

- t means sleep mode (adjustable 1 to 6 minutes)

It is recommended to write down the programmed acuity values.

7. Switch off the projector at the rear of the projector next to the mains switch.  
8. Re-insert the data line to VISUTRON PLUS .  
11. Switch on the projector at the rear of the projector next to the mains switch.

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## 8. Exchange of Lamps

**Attention:** Switch off the unit, pull off mains plug.

- Loosen the knurled screw at the bottom of the projector.
- Lift the upper half of the housing (1).
- Loosen the grounded outlet from the upper half of the housing.
- Pull the black cover (2) on top of the red knurled screws and lift it into direction of the housing edge.

**Attention:** Be careful with the hot glass tube. Prior to removal let it cool down. Make sure that halogen lamps for 6V/20 W are use only. (Ident. No. 123 112 05). False lamps may cause damage to the unit.

- Insert a new halogen lamp. Depress it vertically to the stop. Do not touch the tube with bare fingers.
- Position the cover (2).
- Connect the grounded outlet to the upper part of housing.
- Position the upper housing part (1), mount it to the bottom side of the projector by knurled screw.

## 9. Care

Remove pollution, dust, etc. with a soft cloth damped with water. Make sure that no water penetrates inside the equipment.

Clean the surface of the objective lens with a clean cotton cloth. Damp it with alcohol, if required.

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## 10. Trouble Shooting and Remedy

- Projector cannot be switched on:  
Check power connections.  
Check fuses.
- No illumination:  
Check fuses  
Exchange halogen lamp.
- Blurring image:  
Re-focus via objective lens.
- Remote control fails:  
Exchange the batteries of remote control.  
See page 7

## 11. Technical Data

Projector .....	340 x 190 x 215 mm
Diameter of adapter pivot .....	30 or 47 mm
Weight.....	4.8 kg
Size of image at 5 m distance.....	245 x 225 mm
Distance of projection directly or via mirror.....	4-8 m
Halogen lamp .....	6V/20W
Operating voltage .....	230 VAC
Maximal power consumption .....	40 VA

Infrared remote control .....	175 x 60 x 25 mm
Weight.....	0.14 kg
Batteries.....	4 each 1.5V micro

Projection screen	450 x 450 mm
Weight.....	1 kg

### Operation and Storage Conditions

Operation conditions.....	+ 10 ... + 40 deg. C
Storage conditions.....	- 20 ... + 60 deg. C
Relative air humidity.....	10 ... 90 %
Air pressure .....	600 ... 1300 mbar

Operation in explosion hazard areas not permitted

Due to continuous technical improvements the figures and technical data shown in this manual may not be identical with the equipment delivered to you.

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J.D.MÖLLER Optische Werke GmbH
## **MÖLLER-WEDEL GmbH**